

# REVIVE™

## M2 GEAR PACK — PNEUMATIC CONSOLE QUICK START GUIDE



Scan for complete instructions

**1 POWER SWITCH**  
Used to turn On/Off the M2 pneumatic console

**2 START/STOP BUTTON**  
Use to start or stop treatment. To pause treatment, press start/stop and hold for 3 seconds. Press start/stop again to resume

**3 BATTERY STATUS**  
Indicates how much charge is in battery

**4 TIME BUTTON**  
Used to set length of treatment time

**5 STATUS INDICATOR**

- Power is on
- Treatment in progress
- Error indication

**6 PRESSURE DIAL**  
Light – 20 to 35 mmHg  
**Medium** – 40 to 65 mmHg  
**Intense** – 70 to 100 mmHg

**7 DC ADAPTER SOCKET**  
Used for connecting the universal power adapter

**8 AIR CONNECTION PORTS (2)**  
Used to connect garments to the pneumatic console

**9 AIR CONNECTION BLOCKER PLUG**  
Use blocker plug when only one garment is in use




**1 7**  
(On Back)



# REVIVE<sup>™</sup>

## M2 GEAR PACK — PNEUMATIC CONSOLE QUICK START GUIDE

### USING THE REVIVE M2 PNEUMATIC CONSOLE

1. Place the REVIVE console on a flat stable surface and insert the power adapter into the DC adapter socket on the console to charge the battery.  
**Power Switch must be on to charge the battery**
2. Put on the garment and connect the garment hoses
3. Insert the Blocker Plug to block the unused air outlet for single garment use
4. Set treatment pressure by pressing in and turning pressure knob (Recommended pressure for first time users is 40 mmHg)
5. Press the Time button  to set the treatment time to 15 minutes, 30 minutes, 45 minutes, 60 minutes or continuous mode
6. Select the HOLD Time by holding down the Time button 
  - Short HOLD – Press for 3 seconds (1 beep)
  - Medium HOLD – Release and press again for 3 more seconds (2 beeps)
  - Long HOLD – Release and press again for 3 more seconds (3 beeps)
7. Start the treatment by pressing the Start/Stop button 

### CHARGING THE REVIVE M2 PNEUMATIC CONSOLE

- Turn on the Power Switch to enable charging the battery
- Turn Off the Power Switch to conserve battery power when not charging or not using the REVIVE M2 pneumatic console
- Battery level is continuously displayed by the battery status indicators



Scan for complete instructions

