

REVIVE[™]

M4 GEAR PACK — PNEUMATIC CONSOLE QUICK START GUIDE



Scan for complete instructions and user Bluetooth control apps



1 POWER SWITCH
Used to turn On/Off the M4 pneumatic console

2 MODE/MENU BUTTON
Used to access the selection of a mode, treatment pressure, treatment time, cycle mode, release time, and hold time

3 UP AND DOWN BUTTONS
Used to select a mode, treatment pressure, treatment time, cycle mode, release time, and hold time

4 START/STOP BUTTON
Used to start or stop treatment

5 DC ADAPTER SOCKET
Used for connecting the universal power adapter

6 LCD DISPLAY SCREEN

- Displays pressure setting at the most distal pressure zone
- Pressure may be increased or decreased from this home screen
- Displays remaining treatment time

7 AIR CONNECTION PORTS (2)
Used to connect garments to the pneumatic console

8 AIR CONNECTION BLOCKER PLUG
Use blocker plug when only one garment is in use



REVIVE[™]

M4 GEAR PACK — PNEUMATIC CONSOLE QUICK START GUIDE

USING THE REVIVE M4 PNEUMATIC CONSOLE

1. Place the REVIVE console on a flat stable surface and insert the power adapter into the DC adapter socket on the console to charge the battery. Power Switch must be on to charge the battery
2. Put on the garment and connect the garment hoses
3. Insert the Blocker Plug to block the unused air outlet for single garment use
4. Select the Treatment Mode
 - From the Home screen, press  to select the desired mode
5. Set Treatment Pressure
 - Recommended pressure for first time users is 40 mmHg
 - To select pressure, press  or 
 - To confirm a selection, press 
6. Modify other treatment settings
 - Press and hold  until the menu screen is displayed
 - Select setting you wish to change with  or 
 - Press  and the highlighted value will start blinking
 - Choose preferred value with  or 
 - To confirm a selection, press 
7. Set Gradient
 - Press  and the highlighted zone will start blinking
 - Choose preferred pressure with  or 
 - To confirm a selection, press . Repeat the above steps for all zones
8. Exit the menu with the Start/Stop button 
9. Start the treatment by pressing the Start/Stop button 

CHARGING THE REVIVE M4 PNEUMATIC CONSOLE

- Turn on the Power Switch to enable charging the battery
- Turn off the Power Switch to conserve battery power when not charging or not using the REVIVE M4 Pneumatic Console
- Battery level is continuously displayed by the battery status indicators

USING YOUR MOBILE APP VIA BLUETOOTH CONNECTIVITY

- Choose treatment mode
- Modify treatment parameters
- Start or stop the console using your smartphone
- Retrieve usage history from the console



Scan for complete instructions and user Bluetooth control apps

